

User Manual

For

LED Device



Thanks for using our instrument ! For your fast and sincerely service please read the manual carefully when you use the machine.

Our duty is at your service !

Our pursuit is satisfying you !

Our motto:

Credit based

Quality first

1. Safety Notices

To prevent any potential accident, please operate this device according to the following safety notices.

DO NOT:

- Immerse the LED light in water
- Wash this instrument in water
- Use a wet cloth to wipe down the unit
- Spill liquids in the light chamber
- Open this unit
- Put any items, except syringe and contents to be activated in the light chamber

2. Features and control



Your unit has the following features and controls:

- a. Countdown time digital display
- b. Start-off button
- c. Built-in Fan
- d. Reminding calling for the last 15 second to the end of the whole process

3. How to operate

- a. Place on a flat, dry workstation
- b. Connect to the power.
- c. Turn on switch
- d. Put Syringes into light chamber
- e. Set up the time (Left window display control the first light chamber, right window display control the second light chamber)
- f. Press “Start” to turn on the light.
- e. Wait for the procedure to be completed.
- g. After completing all the process, turn off switch, unplug the instrument from the power outlet.

4. Care and Maintenance

- a. LED light with few serviceable parts, so maintenance is restricted to cleaning of this unit.
- b. Wipe the unit with a soft dry cloth daily after use.
- c. If no parameter showing, please check the power connection is good or not.
- d. Do not open this unit.

5. Technical parameter

1. Microcontroller controlled
2. Two digital window showing, L1 showing L1 light chamber,L2 showing L2 light chamber.
3. After the whole procedure finished, light will be off automatically
4. Sound reminding. From the last 15 second to the end of procedure, the instrument will buzz.
5. Internal syringe holder, including 40 LED light, 10 red, 10 yellow, 10 green,10 Blue. 6. Power supply, 110V 60HZ

6. Warranty Policy

1. We have one –year warranty on our all products. Damaged by misuse and dropping unit is not our guarrantee range.

2. We will pay attention to the feedback of the customers within 24 hours.

3. We won't charge customers any money for repairing the machine in one-year warranty time.

4. We only charge for the accessories on repairing the machine beyond the one-year warranty time.